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Please read the instructions carefully before using for the first time!

WARNING means: failure to follow these instructions may cause damage to the product or injury to the patient or other persons.

1. Specifications and Scope of Supply

Item No.:	4500028	4500029	4500030
Dimensions:	95 x 50 cm foldable: 48 x 50 cm	95 x 50 cm non-foldable	180 x 50 cm foldable: 90 x 50 cm
Material Rollboard: Material cover:	Polyethylene, latex free Sliding fabric with coated silver cover	Polyethylene, latex free Sliding fabric with coated silver cover	Polyethylene, latex free Sliding fabric with coated silver cover
Colour Rollboard: Colour cover:	graphite silver	graphite silver	graphite silver
Roll load:	max. 300 kg	max. 300 kg	max. 300 kg
	X-ray compatible	X-ray compatible	X-ray compatible
Scope supply	Rollboard incl. cover Manual	Rollboard incl. cover Manual	Rollboard incl. cover Manual
	 max. 300 kg		Cover:     

Made in Germany

Basic body

Transatlantic Rollboard Special Edition 30

2. Use/Placement

The Rollboard Special Edition 30 is ideally suited for atraumatic, painless and low-friction transfer of seated and recumbent, e.g. heavy, particularly helpless or premedicated patients to a mobile shower trolley, a different bed, a trolley, different types of wheelchairs, an x-ray, examination or operating table.

Transfer is also possible over a short distance or over different heights. The Rollboard Special Edition 30 is extremely flexible with its support for the transfer of patients with many drainages or with problems such as respiratory distress syndrome. For safety reasons, transfers with the Rollboard Special Edition 30 should be done with two persons and if applicable with a sheet for pulling the patient.

Move the patient in a recumbent position slightly to the side and slide the Rollboard Special Edition 30 about 1/5 of the way under the patient. Place the patient in a supine position on the Rollboard Special Edition 30. Pull the patient over and onto the Rollboard Special Edition 30 in the transfer direction with the pull sheet. Without the pull sheet stand beside the transfer level and push the patient onto the Rollboard Special Edition 30 in the transfer direction.

WARNING:

» Ensure the working heights of the transfer levels are correct for working without back injury.

To remove the Rollboard Special Edition 30 place the patient briefly on the side and remove the Rollboard Special Edition 30 from under the patient.

The Rollboard Special Edition 30 can be used by four carers when transferring very heavy persons. The Rollboard Special Edition 30 can also be briefly lifted with a load during a transfer to negotiate an obstacle. The Rollboard Special Edition 30 can be pulled up or down, moved or briefly lifted to move it away from an unfavourable transfer situation. Always be aware of your back when working as a carer.

This product is suitable for re-use.

Options:

The Rollboard can also be used with Transa gliding foil. The following versions of Transa gliding foil are available:

- » 60 cm × 100 m (item No. 4400200)
- » 60 cm × 100 m in a dispenser box (item No. 4400203)
- » 53 cm × 180 cm, 60 pieces per roll (item No. 4400204)

Cut the appropriate length of Transa gliding foil from the roll (approx. 80 cm to 180 cm – depending on the type of Rollboard). Pull the gliding foil over the Rollboard. In combination with the special edition 30 Rollboard, Transa gliding foil simplifies even difficult transfers, thanks to its outstanding gliding properties.

For application in hygiene-sensitive environments, please use the following disposable protective film:

- » disposable protective film 60 × 190 cm (suitable for Rollboards from 180 cm to 195 cm, item No. 4500120)
- » disposable protective film 60 × 100 cm (suitable for roll boards up to 100 cm, item No. 4500124)

3. Cleaning, Care and Disinfection

The Transatlantic Rollboard Special Edition 30 must be cleaned with medical disinfectants before and after every change of patient.

3.1. Blue bottom cover

Do not use abrasive or bleaching agents or sharp objects for cleaning. The use of bleach- or alcohol-based cleaning agents can permanently impair the functional properties of the blue cover. The blue cover can be easily disinfected using oxygen-based disinfectant solutions or Trionic® (DGHM/VAH approved). For use in hygienically sensitive areas, please use the disposable protective film (item no. 4500120).

3.2. Silver cover

The use of bleaching agents can permanently impair the functional properties of the silver cover. The silver cover can be easily disinfected with alcohol-based disinfectant solutions or with Trionic® (DGHM/VAH approved).

3.3. Board

The board itself is very durable and can be cleaned with standard household cleansing products.

Conventional cleaning: Use warm water and mild detergent. Wipe the product with a damp cloth if necessary and then allow to dry it thoroughly.

Medical cleaning: The Board may easily be disinfected with alcohol-based disinfectants as well as with Trionic® (German DGHM/VAH approval). Allow the board to dry after disinfection.

It is not recommended to use the autoclave due to the thermo-sensitive structure of the material.

Important Note:

» When using the Rollboard Special Edition 30 with the TRANSA disposable sliding foil (the sliding foil is disposable and not washable) please note the special instructions of the sliding foil.

*Option:
Use of the TRANSA disposable
sliding foil*

4. Warranty

The warranty is valid for a period of 3 years. It does not cover any defects which are due to improper use. Transatlantic cannot be liable for any accidental or indirect injury or damage.

5. Visual and Function Inspection/ Safety Information

Repositioning products are also subject to natural aging. Check the equipment regularly and carefully to make sure it is working properly. Make sure that the product is in good condition and that the material and seams are not damaged.

WARNING:

- » Faulty products must no longer be used.
- » Make sure that there are a sufficient number of carers and assistants corresponding to the transfer situation.
- » Never leave the patient unattended.
- » The smooth surface considerably minimises resistance to friction during the change of position. Carefully check the two transfer surfaces, making sure they are stable and brakes are applied (wheelchair, bed).
- » The distance between free transfer surfaces using the Rollboard Special Edition 30 must not be more than 15 cm.
- » Particularity when lifting the Rollboard Special Edition 30 make sure that the patient is securely on the Rollboard Special Edition 30 and cannot slide off.
- » Make sure that carers are always aware of their backs during the transfer.
- » Note the weight limit (300 kg).
- » Do not use sharp-edged items when using the Rollboard Special Edition 30. The Rollboard Special Edition 30 could get damaged in this case.
- » Transfers under uncovered skin must be done with a cloth between the Rollboard Special Edition 30 and the patient (e. g. towel).

6. Product Labeling

The transfer aid Rollboard Special Edition 30 is a medical product, class I and complies with the MDD directive 93/42/EEC. Transatlantic is certified in accordance with the directive DIN EN ISO 13485 by ECM Zertifizierungsgesellschaft für Medizinprodukte in Europa mbH.

7. Disposal Information

For the Rollboard Special Edition 30 there are no special disposal rules; it can be disposed of in accordance with national and local regulations as necessary.

8. Accessories

Transa - Slidefoil

Item-No. 4400200

Optionally available in dispenser box with knife

Item-No. 4400203

Disposable protective foil

Item-No. 4500120

Organizer for Rollboard

Item-No. 4500417

If you have any questions about the use of our products, please call our service number:

+49 (0)6081 9430-50

"Information subject to technical modifications to our products and deviations in colour or printing errors"

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